

Welcome to **lexie**

Let's get started.

Download the Lexie app.

1



Download from the App Store or Google Play Store. The app lets you control your hearing aids and teaches you how to wear them.

Open the battery doors and insert the batteries in them with the stickers facing up and the rounded sides facing down. Then, peel the stickers off and fully close the doors. You should feel two clicks.

2



3



Follow the instructions in the app.

Need help? See reverse for more information.





We're here to help.

Call (779) 212-7334



For more detailed instructions, refer to the “Set up the hearing aids” section in the user manual.



Visit the Lexie product support webpage for videos, downloads, articles, and other resources.

<https://lexiehearing.com/us/support>



©2016-2022 hearX IP (Pty) Ltd.

44-0006-03, REV1